

BICRO™ ELASTIC ANKLE SUPPORT

APPLICATION INSTRUCTIONS



1. Remove support from bag/box
2. Pull the support onto the injured foot in a sock like manner
3. Position the heel opening comfortably over the heel area
4. Re-adjust if necessary

**NOT MADE WITH
natural rubber latex**

Cautions

- Regularly inspect application area during use.
- If you experience skin irritation or increased/persistent pain or discomfort while using this product, discontinue use and consult your healthcare provider.

Reorder No.	Size	Unit
5000 4192	Small	Each
5000 4193	Medium	Each
5000 4194	Large	Each
5000 4195	X-Large	Each

Washing Instructions

Hand wash warm water • Mild detergent • Air dry

Single Patient Use

MANUFACTURED BY

BIRD & CRONIN® INC.

Quality Medical Products To Help Improve Your Life

1200 Trapp Road • Eagan, Minnesota USA 55121

651/683-1111 • Fax 651/688-9855 • 800/328-1095 • Fax 800/279-7934

www.birdcronin.com • ©2015 Bird & Cronin®, Inc. All Rights Reserved

10004540