

Universal Arm Sling

NOTE: Instructions cover multiple product families, images may vary.

Please read the following instructions carefully and completely before use. Correct application is required to ensure proper function of the device.

Intended for single patient use.

NOT MADE WITH natural rubber latex



WARNING: If your pain increases or persists or you experience any unusual reactions while using this product, consult your healthcare provider immediately.

APPLICATION INSTRUCTIONS

1. Slide affected arm into sling pouch (A) and position elbow snugly into the back corner.
2. Size sling pouch (A) by folding the extra length inward. Secure hook tabs to the loop strips (B) located inside the sling pouch (A).
3. Wrap shoulder strap (C) diagonally across your back and over your unaffected shoulder.
4. Guide end of strap (C) through both O-rings (D) on sling pouch (A), loop back and secure.
5. Use hook and loop closures located at both ends of shoulder strap (C) to adjust length.

NOTE: Please refer to sew-in label on device for care instructions.

Please visit our website at www.birdcronin.com for additional product information or contact a customer care representative at **800-328-1095**.

NOTICE: Every effort has been made to obtain the maximum compatibility of function, strength, durability and comfort with this product, however, there is no guarantee that injury will be prevented with its use.

