

# CRADLES ELBOW AND HEEL PROTECTOR APPLICATION INSTRUCTIONS



1. Remove elbow protector from box/bag
2. Slide the protector over elbow or heel positioning the padded portion at the area to be protected
3. Re-adjust if necessary

**NOT MADE WITH  
natural rubber latex**

<b>Reorder No.</b>	<b>Size</b>	<b>Unit</b>
5000 0424	Universal	Pair
0814 0424	Universal	Pair

### **Washing Instructions**

**Hand wash warm water • Mild detergent • Air dry**

**SINGLE PATENT USE**

MANUFACTURED BY

***BIRD & CRONIN® INC.***

---

---

*Quality Medical Products To Help Improve Your Life*

1200 Trapp Road • Eagan, Minnesota USA 55121

651/683-1111 • Fax 651/688-9855 • 800/328-1095 • Fax 800/279-7934

www.birdcronin.com • ©2015 Bird & Cronin®, Inc. All Rights Reserved

10000400