

BIRD & CRONIN[®] INC.

Improving lives with quality medical products

Revere Lumbosacral Support

Please read the following instructions carefully and completely before use. Correct application is required to ensure proper function of the device.

Intended for single patient use.

NOT MADE WITH natural rubber latex



WARNING: If your pain increases or persists or you experience any unusual reactions while using this product, consult your healthcare provider immediately.



Figure 1



Figure 2



Figure 3

APPLICATION INSTRUCTIONS

1. Wrap support around waist like a belt. Center contoured stays on the lower back and position for a comfortable fit.
2. Grip both ends of the support and stretch evenly.
3. Pull the left end across the abdomen and hold firmly against right side (Figure 1).
4. Pull right end across on top and secure to left side of abdomen.
5. Grip the ends of both elastic side straps and stretch evenly (Figure 2).
6. Pull and secure the left strap to the right side of abdomen.
7. Pull and secure the right strap to the left side of abdomen.
8. Adjust side pulls as needed to modify level of tension, support, or comfort.

NOTE: Please refer to sew-in label on device for care instructions.

Please visit our website at www.birdcronin.com for additional product information or contact a customer care representative at **800-328-1095**.

NOTICE: Every effort has been made to obtain the maximum compatibility of function, strength, durability and comfort with this product, however, there is no guarantee that injury will be prevented with its use.



Bird & Cronin, Inc
1200 Trapp Road · Eagan, MN USA 55121
800-328-1095 · www.birdcronin.com