

BIRD & CRONIN[®] INC.

Improving lives with quality medical products

Knee Strap

NOTE: Instructions cover multiple product families, images may vary.

Please read the following instructions carefully and completely before use. Correct application is required to ensure proper function of the device.

Intended for single patient use.

NOT MADE WITH natural rubber latex



WARNING: If your pain increases or persists or you experience any unusual reactions while using this product, consult your healthcare provider immediately.

APPLICATION INSTRUCTIONS

NOTE: Application may be done either standing or sitting.

1. Extend leg out and relax thigh muscle. Position the strap just below the knee cap.
2. Secure the strap behind the knee with the hook and loop closure.
3. Adjust as needed for compression and comfort.
4. Strap may be trimmed after initial fitting if desired for less bulk. Trim the end without the hook tab and position the inside tab to the medial (inside) or lateral (outside) side of the knee.

CARE INSTRUCTIONS

Fasten hook-and-loop together. Hand wash with cold water and mild detergent. Air dry.

Please visit our website at www.birdcronin.com for additional product information or contact a customer care representative at **800-328-1095**.

NOTICE: Every effort has been made to obtain the maximum compatibility of function, strength, durability and comfort with this product, however, there is no guarantee that injury will be prevented with its use.



Bird & Cronin, Inc
1200 Trapp Road · Eagan, MN USA 55121
800-328-1095 · www.birdcronin.com

©Bird & Cronin[®], Inc. 10008020