

AIR TRIO® Shell Ankle Walker

NOTE: Instructions cover multiple product families, images may vary.



Figure 1



Figure 2



Figure 3



Figure 4



Air Trio® Shell
Ankle Walker Tall



Air Trio® Shell Ankle
Walker Low Top

**NOT MADE WITH
natural rubber latex**

Please read the following instructions carefully and completely before use. Correct application is required to ensure proper function of the device.

Intended for single patient use.



WARNING: If your pain increases or persists or you experience any unusual reactions while using this product, consult your healthcare provider immediately.

APPLICATION INSTRUCTIONS

NOTE: High pressure in the bladders provides more support when walking; lower pressure is more comfortable when sitting or reclining. Pressure can be adjusted by inflating/deflating the air bladders or tightening/loosening the device straps.

NOTE: The Air Trio® should be worn with the Comfor® Max Walker sock. Additional pads included if needed.

1. Unfasten straps, remove anterior panel, and open liner. Remove additional pads and bulb located inside boot and set aside.
2. Place the Comfor® Max Walker sock on the patient if desired.
3. While seated, place leg in brace and position heel at the back of the walker. Wrap liner around the leg and foot and secure.
4. Attach the toe cover by folding it over the toes and securing the hook to the liner. Cover can be removed if desired. (Figure 1)
5. Replace the anterior panel over the liner. (Figure 2)
6. Beginning with the toe strap, feed the straps through the links and work up the leg by securing each strap. (Figure 3)

NOTE: Pads may be inserted at either side of the ankle or at heel for additional comfort.

NOTE: Liner and strap adjustment may be necessary as swelling patterns change throughout the rehabilitation process.

NOTE: Tall ankle walker has three leg straps and the low top ankle walker has one leg strap.

NOTE: The inflation tubes can be re-positioned on the liner or removed if desired.



WARNING: Do not over-inflate the air bladders. Improper inflation may cause skin irritation and/or damage to the device.

7. Inflate the air chambers, located on sides of the walker, by placing the CLEAR end of the inflation bulb into the air tubes and pump with one hand. Both air chambers should be inflated to provide even compression. (Figure 4)
8. To release air from the chambers, place the BLACK end of the inflation bulb into the air tubes and pump to deflate.

NOTE: Please refer to sew-in label on device for care instructions.

Please visit our website at www.birdcronin.com for additional product and warranty information or contact a customer care representative at 800-328-1095.

NOTICE: This device is only one element in the overall treatment program administered by a medical professional. While every effort has been made to obtain the maximum compatibility of function, strength, durability and comfort, there is no guarantee that injury will be prevented through the use of this product.