

## TK Thumb Orthosis

Please read the following instructions carefully and completely before use. Correct application is required to ensure proper function of the device.

**Intended for single patient use.**



**WARNING:** If your pain increases or persists or you experience any unusual reactions while using this product, consult your healthcare provider immediately.



### APPLICATION INSTRUCTIONS

1. Unfasten straps and slide affected thumb into brace.
2. Wrap and pull wrist strap (B) to desired compression and secure.
3. Wrap and secure thumb strap (A) when desired fit and level of compression is reached.
4. Adjust straps as needed when swelling of the hand, wrist or thumb changes.

**NOTE:** Interchangeable flexible, rigid and malleable stays can be removed if desired.

**NOTE:** Please refer to sew-in label on device for care instructions.

Please visit our website at [www.birdcronin.com](http://www.birdcronin.com) for additional product information or contact a customer care representative at **800-328-1095**.

**NOTICE:** Every effort has been made to obtain the maximum compatibility of function, strength, durability and comfort with this product, however, there is no guarantee that injury will be prevented with its use.

**NOT MADE WITH  
natural rubber latex**

