

## Prody™ Abdominal Binder with Drain Fasteners

Designed to securely hold suction reservoirs during post-operative recovery.



U.S. Patent #6,610,032B1

Please read the following instructions carefully and completely before use. Correct application is required to ensure proper function of the device.

**Intended for single patient use.**



**WARNING:** If your pain increases or persists or you experience any unusual reactions while using this product, consult your healthcare provider immediately.

### APPLICATION INSTRUCTIONS

1. Position the binder between the lower rib area and hip to cover the abdominal area.
2. Secure the hook and loop closure with enough stretch to provide comfortable support to the abdominal area.
3. Lift blue hook locking tab from binder and thread through loop attached to drainage bulb. Press hook to secure. Repeat as necessary for up to four drainage bulbs.
4. Re-adjust as needed for comfort.

**NOTE:** Please refer to sew-in label on device for care instructions.

Please visit our website at [www.birdcronin.com](http://www.birdcronin.com) for additional product information or contact a customer care representative at **800-328-1095**.

**NOTICE:** Every effort has been made to obtain the maximum compatibility of function, strength, durability and comfort with this product, however, there is no guarantee that injury will be prevented with its use.

**NOT MADE WITH  
natural rubber latex**

