

L'TIMATE® Hinged Knee with Universal Lateral Pull



Figure 1



Figure 2



Figure 3



Figure 4

**NOT MADE WITH
natural rubber latex**

Please read the following instructions carefully and completely before use. Correct application is required to ensure proper function of the device.

Intended for single patient use.



WARNING: If your pain increases or persists or you experience any unusual reactions while using this product, consult your healthcare provider immediately.

APPLICATION INSTRUCTIONS

1. Open sleeve and pull onto leg. Ensure patella opening is centered over the kneecap. (Figure 1)

NOTE: The optional lateral pull strap may be removed and placed on either side of the sleeve.

2. Once sleeve is positioned as desired, secure optional lateral pull strap to side of the sleeve with hook and loop. (Figure 2)

3. Pull straps across the patella buttress and secure with hook. (Figure 3)

4. Adjust straps to achieve desired fit and level of compression.

NOTE: The tubular patella buttress can be modified if desired by cutting and removing portions through the slits on the reverse side of sleeve. Medial/Lateral hinges may be removed if desired through the slits on the reverse side of the sleeve.

NOTE: Please refer to sew-in label on device for care instructions.

Please visit our website at www.birdcronin.com for additional product information or contact a customer care representative at **800-328-1095**.

NOTICE: Every effort has been made to obtain the maximum compatibility of function, strength, durability and comfort with this product, however, there is no guarantee that injury will be prevented with its use.

