

CATH-MATE® II


Catheter Holder - *Latex Free*

U.S. Patent #5,352,209

Reorder Number

0814-8214

PROPER APPLICATION ILLUSTRATIONS

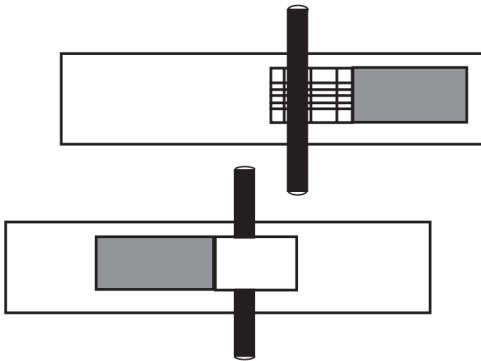

1. Position and secure leg strap to patients thigh.
2. Align and position catheter on patented grip securing tube strap.
3. Slightly stretch elastic and secure strap across the catheter and fasten to leg strap. Catheter is now firmly secured and should not slip.

BIRD & CRONIN INC.

Quality Medical Products To Help Improve Your Life
1200 Trapp Road • Eagan, Minnesota USA 55121
651/683-1111 • Fax 651/688-9855
800/328-1095 • Fax 800/279-7934