

Cervical Collar

Please read the following instructions carefully and completely before use. Correct application is required to ensure proper function of the device.

Intended for single patient use.



Figure 1



Figure 2

**NOT MADE WITH
natural rubber latex**



WARNING: If your pain increases or persists or you experience any unusual reactions while using this product, consult your healthcare provider immediately.

APPLICATION INSTRUCTIONS

1. Position the collar contour under the chin. Hook and Loop should be to the outside of the collar.
2. Wrap around to the back of neck and secure with hook and loop closure. (Figure 2)
3. Adjust as needed for support or comfort.

NOTE: Please refer to sew-in label on device for care instructions.

NOTICE: Every effort has been made to obtain the maximum compatibility of function, strength, durability and comfort with this product, however, there is no guarantee that injury will be prevented with its use.

