

Bicro™ Elastic Ankle Support

Please read the following instructions carefully and completely before use. Correct application is required to ensure proper function of the device.



Intended for single patient use.

APPLICATION INSTRUCTIONS

1. Pull the support onto the injured foot in a sock like manner.
2. Position the heel opening comfortably over the heel area.
3. Adjust as needed for comfort.

NOTE: Please refer to sew-in label on device for care instructions.

Please visit our website at www.birdcronin.com for additional product information or contact a customer care representative at **800-328-1095**.

NOTICE: Every effort has been made to obtain the maximum compatibility of function, strength, durability and comfort with this product, however, there is no guarantee that injury will be prevented with its use.

**NOT MADE WITH
natural rubber latex**

